

RC-Blue



www.dental-perfect.com



RECIPROCATING SYSTEM

High efficiency and
cutting performance
achiever

Special design in apical
Favorable helical
Non cutting tip


Size	Taper	Material: Blue NiTi 
R25	.08	Length: 21mm,25mm,31mm
R40	.06	Rotary speed: 350rpm
R50	.05	Package: 3 pcs or 6 pcs per box/blister, assorted/single size




RECIPROCATING MOVEMENT PATTERN

- Preparation of the root canal with a single file is convenient and quick
- Less working steps, more efficiency and less time-consuming
- Suitable for curved and narrow root canal preparation
- New NiTi material enhances the flexibility and ability of anti-fracture
- Three sizes are available for different types of root canals

SPECIAL DESIGN IN APICAL

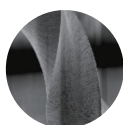


FAVORABLE HELICAL

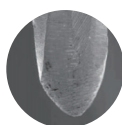

NON CUTTING TIP

- © 2 main cutting-edge designs of engines use nickel-titanium equipment.
- © It has a cutting apical triangular cone and cutting edge without cutting effect, which is designed in conical.
- © The unique apical design, which is a non-consider cutting function but supervises and guides the device, reduces the risk of steps and fracture.

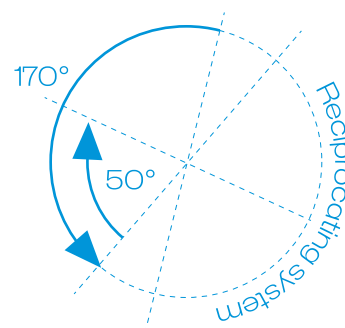

OPERATION GUIDE:

Taking photos of the root canal tip before root canal preparation so that it can display the canal diameters lengths, root canal numbers and the bending of the root canal.

FOR NARROW OR INVISIBLE CANAL

Opening the pulp, searching the root canal, using the #10 or #15 (stainless steel K file) to the working length.

The R25 RC-Blue File is most suitable in the great majority of cases.



R25 .08

#30 K file can reach the working length passively.

Yes



R50 .05

#20 K file can reach the working length passively.

Yes



R40 .06

No



R25 .08

RC-BLUE OPERATING CASES:



CE 0197